



Specimen Requirements

Recurrent Pregnancy Loss Panel (Page 1 of 2)

TESTS	SPECIMEN	VACUTAINER TUBE	STORAGE TEMP	SHIPPING TEMP
<ul style="list-style-type: none"> • Anti-Phospholipid Antibodies Panel (IgG & IgM) • Anti-Nuclear Antibodies Panel • Anti-Thyroid Antibodies Panel 	Serum	Tiger SST or Red spin & separate	Refrigerate 2-6° C	Ambient
<ul style="list-style-type: none"> • Coagulation Panel 	Plasma	Blue *	Frozen -20° C	Frozen -20° C
<ul style="list-style-type: none"> • Reproductive Immunophenotype 	Whole Blood	Lavender Top	Ambient	Ambient
<ul style="list-style-type: none"> • Natural Killer Cells 	Whole Blood	Green Top	Ambient	Ambient

* NOTE: For the Blue Plasma sample, centrifuge immediately at 3500 rpm for 15 minutes. Remove plasma, aliquot into plastic transport tube or cryovial and freeze immediately.



Specimen Requirements

Recurrent Pregnanc Loss Panel (Page 2 of 2)

TESTS	SPECIMEN	VACUTAINER TUBE	STORAGE TEMP	SHIPPING TEMP
<ul style="list-style-type: none"> • TH1 • TH2 	Whole Blood	Green Top	Ambient	Ambient
<ul style="list-style-type: none"> • Karyotyping 	Whole Blood	Green Top	Ambient	Ambient